



**NEW 100%
Recyclable Pack**

www.bossklein.com

Bossklein®

**V-WIPE ZERO
ECO-XL**

Surface Disinfectant Wipe

BIODEGRADABLE†

Plastic free wipe

Plant based fibres

NEW fully recyclable pack

NEW moisture lock flap

Reduced carbon footprint

Alcohol free



KEY FEATURES

V-WIPE ZERO ECO-XL wipes are plastic free wipes made from renewable and sustainable natural viscose fibers, now available in fully recyclable packaging.

Developed for use in clinical environments on non invasive medical devices and general high traffic non-porous surfaces. ECO-XL is a modern and eco friendly alternative to traditional wipes which are made from Polypropylene or Polyester.

The unique ECO-XL wipe substrate material is constructed from plant based viscose fibres which are 100% renewable and sustainably sourced. Wipes are biodegradable† and totally plastic free.

NEW for 2026: Packs are now made from a 100% recyclable material that can be easily recycled alongside other items. Packs also contain a new easy peel, moisture lock flap to further reduce overall plastic consumption.

The alcohol free formulation is ideal to use on the majority of wipeable surfaces even those that are sensitive to alcohol, including dental chairs*

WHAT YOU NEED TO KNOW

- Biodegradable†. Plastic free wipes
- NEW Recyclable packaging
- 100 extra large wipes
- Quick contact time
- Dual registration; Medical Device and Biocide
- Phase II Step I & II tested
- Safe to use on sensitive surfaces*
- Subtle apple aroma

Bossklein ECO-XL contains a synergistic blend of two powerful biocides, is free from PHMB and contains detergent derived from renewable resources.

The formula takes advantage of modern, safer and greener chemical components to create a more simple and effective product for the end user.

V-WIPE Zero ECO-XL wipes are ideal to use where time is critical, such as in between patients and in high traffic areas.

COMPOSITION

Contains Aqua, Alkyl (C12-16) dimethylbenzyl ammonium chloride (ADBAC/BKC (C12-16)) (68424-85-1) 0.48% w/w (4.8g/kg), Didecylidimethylammonium Chloride (7173-51-5) 0.22% w/w (2.2g/kg), Perfume (contains Limonene, Dimethyl Phenethyl Acetate, Hexamethylindanopyran, Menthol).

INSTRUCTIONS FOR USE

Using sufficient wipes for each application wipe the clean surfaces to be disinfected so that they are completely moistened and let the disinfectant take action over 2 minutes. Allow to air dry or wipe with clean tissue. Contains detergent. For professional use only.

Use biocides safely. Always read the label and product information before use.

† Wipes are made from biodegradable plant fibres

*Always test an inconspicuous area first See website for full details.
Do not use on copper or aluminium.

Complete hygiene solutions



Topdental (Products) Ltd
Unit 1, Holmfield Industrial Estate
Halifax, West Yorkshire HX2 9TN
Tel +44 (0)1535 652 750 www.topdental.co.uk

v1.2 0326

Bossklein®

V-WIPE ZERO ECO-XL

- ✓ BACTERICIDAL
- ✓ YEASTICIDAL
- ✓ VIRUCIDAL*



Effective against

This highly effective formula has been rigorously tested to real world EN standards for proven effectiveness including Phase II Steps I and II.

Effective against all listed micro-organisms. This ensures complete peace of mind when it comes to clinical surface disinfection.

The following test standards have been achieved: EN16777, EN16615, EN14476, EN13727, EN13624.

Bactericidal

<i>Pseudomonas aeruginosa</i> , <i>Staphylococcus aureus</i> , <i>Enterococcus hirae</i>	2 mins
--	--------

Yeasticidal

<i>Candida albicans</i>	2 mins
-------------------------	--------

Enveloped viruses

Vaccinia virus, Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus (HDV), HIV, RSV, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella virus, Rabies virus, Measles virus, Poxviridae	60 sec
---	--------



NEW easy to use moisture lock flap



EASY peel and reseal flap reduces total plastic



NEXT wipe remains fresh and ready to dispense

Plant based fibres from certified renewable sources

Wipes are constructed from natural fibres. After use, the plastic free biodegradable† wipes will naturally breakdown overtime.

The construction and large size of the 40gsm, viscose wipe material is extra durable and ideal for clinical disinfection. The plastic free substrate is designed to help transfer the alcohol free disinfectant solution on to the surface in question.



This product is made of material from well-managed, FSC®-certified forests and other controlled sources. For more information see lenzing.com and livabybirlacellulose.com

The Forest Stewardship Council® is an international nongovernmental organization that promotes environmentally appropriate, socially beneficial, and economically viable management of the world's forests. To learn more, visit www.fsc.org



Now with fully Recyclable Pack



NEW for 2026: Each pack of Bossklein V-WIPE ZERO ECO-XL wipes is now 100% recyclable.

Packs are constructed from a matt OPP monofilm laminate that can be recycled under plastic recycling ID code 5 (PP). This means used packs no longer need to be discarded in general waste and end up in landfill.

Packs can now be easily recycled alongside other common items.

BOS8001 Flowpack of 100 ECO-XL Wipes x 9 per box

† Wipes are made from biodegradable plant fibres

* Virucidal against all enveloped viruses

** Always test an inconspicuous area first. Do not use on copper or aluminium

