

Clinician's Choice Diamond Polishers - Instructions For Use

INTENDED USE: Dental finishing and polishing points intended as a medical device for the purpose of final polishing on composite, RMGI, and some ceramic restorations.

INSTRUCTIONS FOR USE: A.S.A.P. Diamond Polishers provide a high shine in two steps with only one shape.

STEP 1: A.S.A.P. Diamond Pre-Polisher (purple)

1. Use a finishing diamond/carbide to contour the composite surface.
2. At 10,000 – 12,000 RPM, using light pressure and intermittent water spray 50mL/min (or damp tooth), apply the pre-polisher to the restorative surface. A smooth silky surface will appear in approximately 20 – 30 seconds.

STEP 2: A.S.A.P. Diamond Final High Shine Polisher (peach)

1. Again, using light pressure and intermittent water spray 50mL/min (or dam tooth), apply the Final High Shine Polisher to the restorative surface in a wiping motion, until a high gloss finish is achieved.

NOTE: Excessive pressure on the instrument will lead to overheating and premature degradation of the polisher, as well as possible damage to the pulp.