

Clinicians Choice Groovy Diamond Polishing Brushes Instructions For Use

INTENDED USE:

These dental polishing and finishing points are intended to be used as a medical device for the purpose of final polishing of all composite and ceramic dental restorations.

INSTRUCTIONS FOR USE:

- Prior to polishing with Groovy Diamond Polishing Brushes, use a fine diamond to refine the restorative surface.
- For best results, use slight to moderate pressure in a wiping or circular motion.
- Use Groovy Diamond Polishing Brushes at 1,000 – 3,000 rpm (maximum) for the ultimate performance. Exceeding the prescribed rpm will reduce the life of the brush without adding improved performance.
- Groovy Diamond Polishing Brushes may be used with or without a water spray.
- There is no need to use a diamond polishing paste or lubricant in conjunction with Groovy Diamond Polishing Brushes.

RECOMMENDED STERILIZATION:

Groovy Diamond Polishing Brushes can be autoclaved up to 135°C OR 275°F. Chemiclave and cold soak sterilization can also be used; however, the product's life may be reduced. Not sold in sterile condition.